

# Animas<sup>®</sup> Vibe<sup>™</sup> CGM Important Codes

**ERR 0**

The sensor is not calibrating correctly.

Wait at least 15 minutes and then enter at least one more blood glucose calibration value.

**ERR 1**

The sensor is not calibrating correctly.

Wait at least 1 hour and then enter in two more blood glucose calibration values.

**BG**

BG value needed for calibration.

**???**

The receiver does not understand the sensor signal temporarily. Wait for more prompts and do not enter any BG values while this symbol is displayed.

Usually resolves of its own accord within 30 minutes.

**ANT**

No communication in the last 5 minutes. Pump and sensor/transmitter are not communicating.

Make sure the pump and sensor/transmitter are within 3.6m of each other and there is no obstruction over the top of the transmitter eg tape.




**HIGH**

Most recent CGM reading >22.2mmol/l.

**LOW**

Most recent CGM reading < 2.2mmol/l.

# Animas Vibe CGM Menu Map

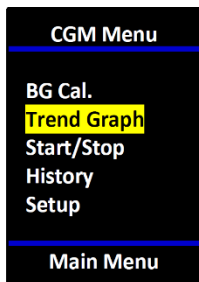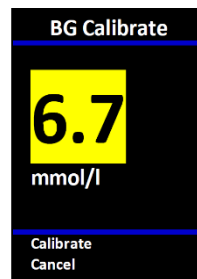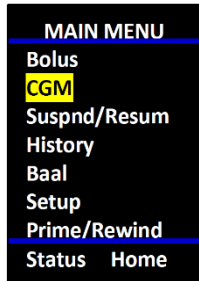

Also graphs for  
1 hour  
3 hour  
6 hour  
12 hour  
24 hours

