

Animas[®] Vibe[™] Pump Menu Map

MAIN MENU

- Bolus**
- CGM
- Suspnd/Resum
- History
- Basal
- Setup
- Prime/Rewind
- Status Home



BOLUS MENU

- Normal**
- ezCarb
- ezBG
- Combo Bolus
- Main Menu

MAIN MENU

- Bolus
- CGM
- Suspnd/Resum**
- History
- Basal
- Setup
- Prime/Rewind
- Status Home



Pump Delivery

- Suspend**
- Main Menu

MAIN MENU

- Bolus
- CGM
- Suspnd/Resum
- History**
- Basal
- Setup
- Prime/Rewind
- Status Home



HISTORY MENU

- Bolus**
- Total Daily Dose (TDD)
- Alarm
- Prime
- Suspend
- Basal
- Main Menu

MAIN MENU

- Bolus
- CGM
- Suspnd/Resum
- History
- Basal**
- Setup
- Prime/Rewind
- Status Home



BASAL MENU

- Total x.xxu
- Temp**
- A1-WEEKDAY
- 2-other
- 3-weekend
- 4-exercise
- Main Menu

Advanced Set Up Menu Map

MAIN MENU

- Bolus
- CGM
- Suspnd/Resum
- History
- Basal
- Setup**
- Prime/Rewind

Status Home



SETUP

- Time/Date
- Sound
- Advanced**

<<-

SETUP ADV 1

- I:C Ratio
- ISF
- BG Target

Home <- !

SETUP ADV 2

BOLUS

- Adv. Bolus **ON**
- Reminders **OFF**
- Delivery **NRML**

BASAL

- Programs **4**

Home <- !->

SETUP ADV 3

MAX LIMITS

- Basal **x.xxU/hr**
- Bolus **x.xxU**

- Daily **xx.xxU**
- 2Hr **xx.xU**

Home <- !

SETUP ADV 4

- Language
- ENGLISH**
- Display
- Timeout
- in **xx** secs
- Contrast

>

Battery Lith

SETUP ADV 5

AUTO OFF

- Enable **OFF**
- When enabled, suspend in **xx** Hr if no buttons are pressed.

Home <- !->

SETUP ADV 6

- Low Cartridge Warning **xxU**
- Occlusion Sensitivity **H**

Home <- !

SETUP ADV 7

AUDIO BOLUS

- Enable **OFF**
- Delivery Step **x.xU**

Home <- !

SETUP ADV 8

Insulin on Board

- IOB-2 **ON**
- Duration

x.xHr

SETUP ADV 9

Sick Days

- BG over **xx.x** mmol/l
- Check ketones
- Every **x** Hrs
- Check BG
- Every **x** Hrs

Home <- !->

Green denotes changeable values