


View Status


for the t:slim X2 Insulin Pump



The instructions below are provided as a reference tool for caregivers who are already familiar with the use of an insulin pump and with insulin therapy in general. Not all screens are shown. For more detailed information on the operation of your Tandem insulin pump, please refer to its user guide.

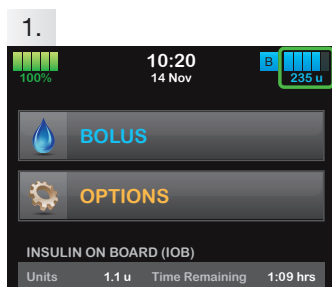


1. Tap the insulin level icon in the upper right corner of the Home Screen.

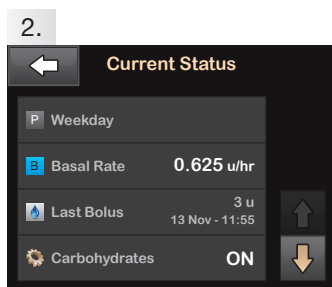


2. Your t:slim X2 pump will display the name of your active profile, your current basal rate, the time and amount of your last bolus, and whether or not you are entering boluses based on carbohydrates.

If you have an active Extended Bolus, this screen will show how much insulin has been delivered out of the total amount requested.

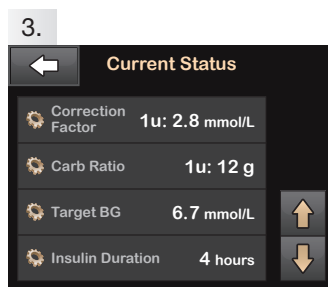


3. Tap the **Down Arrow** to display your current Correction Factor, Carb Ratio, Target BG, and Insulin Duration.

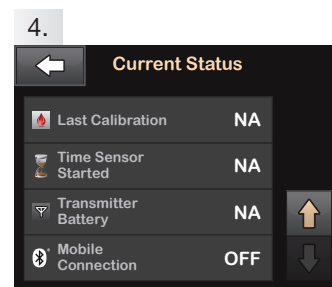

4. Tap the **Down Arrow** to display your Last Calibration, Time Sensor Started, Transmitter Battery status, and Mobile Connection status.

Explanation of Icons

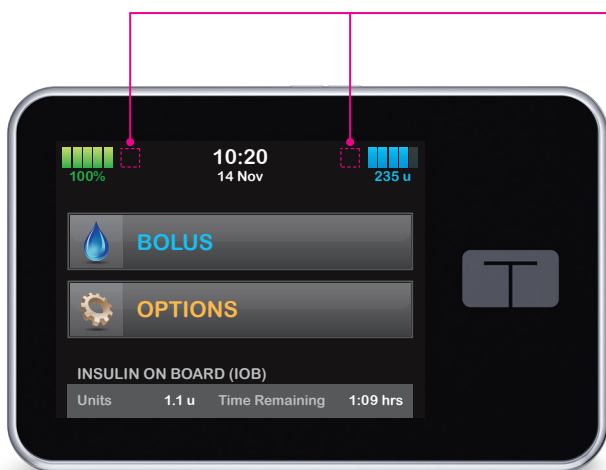

The following icons may appear in the areas to the left or right of the time and date on your t:slim X2 pump's Home Screen.



A system reminder, alert, error, or alarm is active. Unlock screen to view.



A bolus is being delivered.



All insulin deliveries are stopped.



Basal insulin is programmed and being delivered.



A basal rate of 0 u/hr is active.



A temporary basal rate is active.



A temporary basal rate of 0 u/hr is active.



24/7 Technical Support
0508 634 103



t:simulator App

A demo as simple as our pumps!



© 2019 Tandem Diabetes Care, Inc. All rights reserved. Tandem insulin pumps are intended for the subcutaneous delivery of Humalog or NovoRapid/NovoLog U-100 insulin, at set and variable rates, for the management of diabetes mellitus in persons requiring insulin. The t:slim X2™ insulin pump is approved for individuals 6 years of age and older. For important safety information, please visit www.tandemdiabetes.com/safetyinfo. Tandem Diabetes Care is a trademark of Tandem Diabetes Care, Inc., registered in the U.S. and other countries. t:slim X2 is a trademark of Tandem Diabetes Care, Inc. Apple and the Apple logo are trademarks of Apple Inc., registered in the United States and other countries. App Store is a service mark of Apple Inc. Google Play and the Google Play logo are trademarks of Google LLC. Tandem is distributed in New Zealand by NZMS Diabetes, 2a Fisher Crescent, Mt Wellington, Auckland 1060. Ph: 09 259 4062 . April 2020 NZDI_TAN_008_B

For more information on NZMS Diabetes product range, please contact us on
0800 500 226 or at nzms@nzms.co.nz

nzmsdiabetes.co.nz

