


# Set a Temp Basal Rate

## for the t:slim X2 Insulin Pump




The instructions below are provided as a reference tool for caregivers who are already familiar with the use of an insulin pump and with insulin therapy in general. Not all screens are shown. For more detailed information on the operation of your Tandem insulin pump, please refer to its user guide.

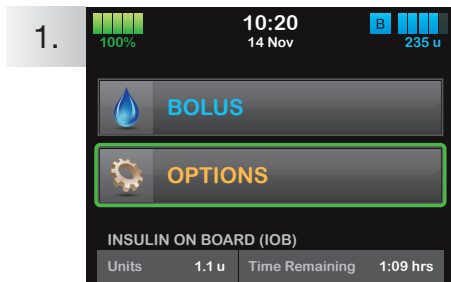


1.

Tap **OPTIONS**.

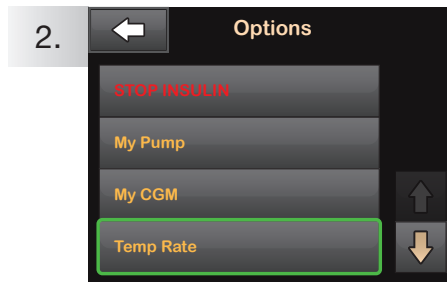


2.

Tap **Temp Rate**.

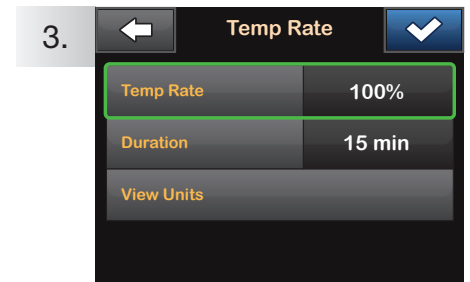

3.


Tap **Temp Rate**.

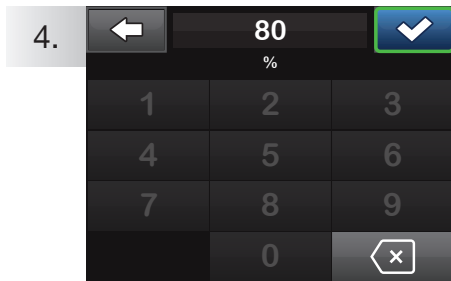

4.


Using the onscreen keypad enter desired percentage. Tap .

**NOTE:** Current rate is 100%. An increase is greater than 100% and a decrease is less than 100%.

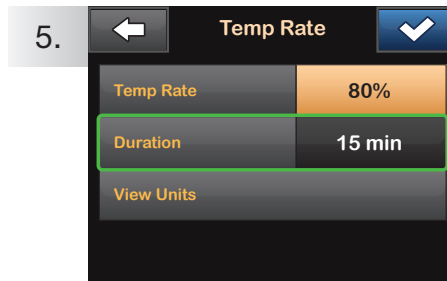

5.

Tap **Duration**. Using the onscreen keypad enter desired length of time for Temp Rate. Tap .

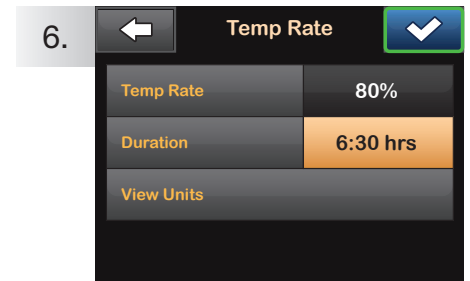

6.


Verify settings and tap .

**NOTE:** To see the actual units to be delivered, tap View Units.



7.


The **TEMP RATE STARTED** screen will appear to confirm the Temp Rate has started.

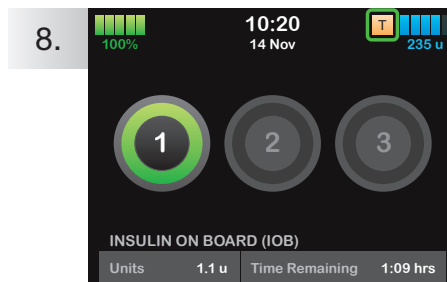

8.

The Screen Lock screen will appear with the orange T icon indicating a Temp Rate is active.

**NOTE:** If a Temp Rate of 0% is currently active, the orange T icon will be replaced with a red T icon.

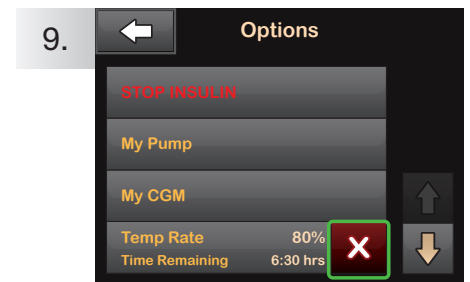

9.

To stop Temp Rate at any time, tap **OPTIONS**, then tap . A confirmation screen will appear. Tap .



**TANDEM**  
DIABETES CARE

24/7 Technical Support  
0508 634 103



**t:simulator** App

A demo as simple as our pumps!



© 2019 Tandem Diabetes Care, Inc. All rights reserved. Tandem insulin pumps are intended for the subcutaneous delivery of Humalog or NovoRapid/NovoLog U-100 insulin, at set and variable rates, for the management of diabetes mellitus in persons requiring insulin. The t:slim X2™ insulin pump is approved for individuals 6 years of age and older. For important safety information, please visit [www.tandemdiabetes.com/safetyinfo](http://www.tandemdiabetes.com/safetyinfo). Tandem Diabetes Care is a trademark of Tandem Diabetes Care, Inc., registered in the U.S. and other countries. t:slim X2 is a trademark of Tandem Diabetes Care, Inc. Apple and the Apple logo are trademarks of Apple Inc., registered in the United States and other countries. App Store is a service mark of Apple Inc. Google Play and the Google Play logo are trademarks of Google LLC. Tandem is distributed in New Zealand by NZMS Diabetes, 2a Fisher Crescent, Mt Wellington, Auckland 1060. Ph: 09 259 4062 . April 2020 NZDI\_TAN\_011\_B.

For more information on NZMS Diabetes product range, please contact us on **0800 500 226** or at [nzms@nzms.co.nz](http://nzms@nzms.co.nz)

[nzmsdiabetes.co.nz](http://nzmsdiabetes.co.nz)

