


# Stop and Resume Insulin Delivery

## for the t:slim X2 Insulin Pump




The instructions below are provided as a reference tool for caregivers who are already familiar with the use of an insulin pump and with insulin therapy in general. Not all screens are shown. For more detailed information on the operation of your Tandem insulin pump, please refer to its user guide.


### ✗ Stop Insulin Delivery

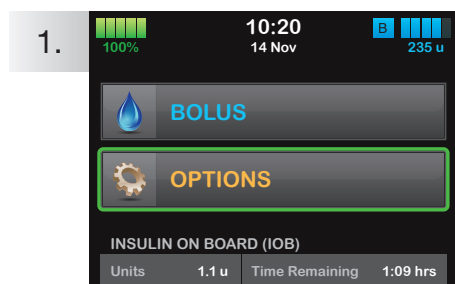

1. Tap **OPTIONS**.

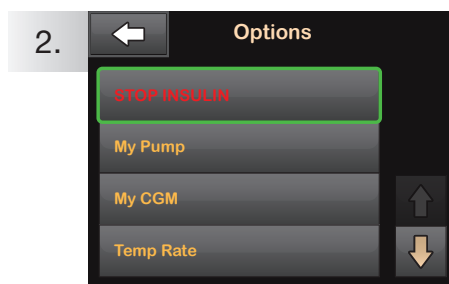


2. Tap **STOP INSULIN**.

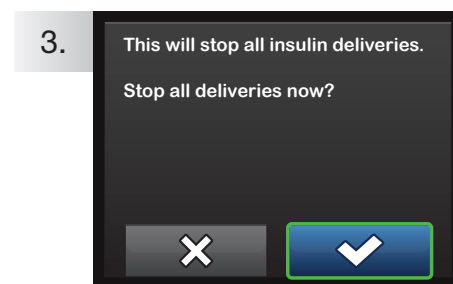


3. Tap


### ✓ Resume Insulin Delivery

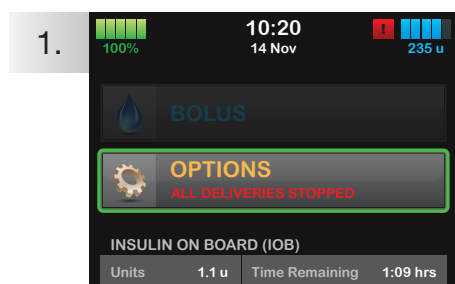

1. Tap **OPTIONS**.

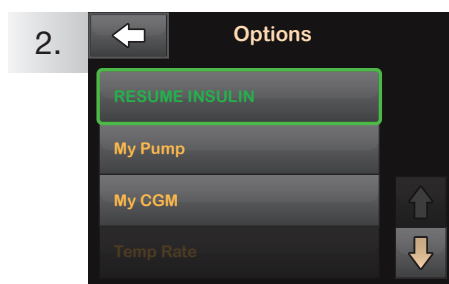


2. Tap **RESUME INSULIN**.

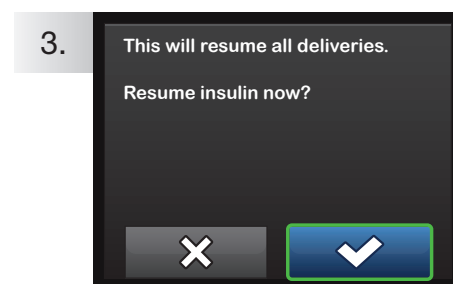

3. Confirm by tapping

**NOTE:** If the timer of a temp rate is still active when insulin is resumed, the temp rate will continue. If any bolus is being delivered before you stop insulin, the remaining insulin to be delivered will be canceled.

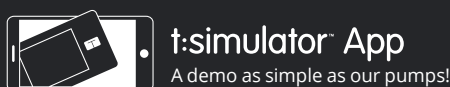

© 2019 Tandem Diabetes Care, Inc. All rights reserved. Tandem insulin pumps are intended for the subcutaneous delivery of Humalog or NovoRapid/NovoLog U-100 insulin, at set and variable rates, for the management of diabetes mellitus in persons requiring insulin. The t:slim X2™ insulin pump is approved for individuals 6 years of age and older. For important safety information, please visit [www.tandemdiabetes.com/safetyinfo](http://www.tandemdiabetes.com/safetyinfo). Tandem Diabetes Care is a trademark of Tandem Diabetes Care, Inc., registered in the U.S. and other countries. t:slim X2 is a trademark of Tandem Diabetes Care, Inc. Apple and the Apple logo are trademarks of Apple Inc., registered in the United States and other countries. App Store is a service mark of Apple Inc. Google Play and the Google Play logo are trademarks of Google LLC. Tandem is distributed in New Zealand by NZMS Diabetes, 2a Fisher Crescent, Mt Wellington, Auckland 1060. Ph: 09 259 4062 . April 2020 NZDI\_TAN\_013\_B.

For more information on NZMS Diabetes product range, please contact us on **0800 500 226** or at [nzms@nzms.co.nz](mailto:nzms@nzms.co.nz)  
[nzmsdiabetes.co.nz](http://nzmsdiabetes.co.nz)

