



# Diasend PDF Wizard Guide

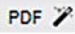

Diasend makes it simple to download and consolidate your patient's diabetes management data in the form of a PDF. For an extensive download of multiple charts, graphs and data, click on the PDF Wizard button  in the top right corner and follow the instructions below.

- 1 Set the end date of the desired time period for the data you wish to obtain.
- 2 Select as many or as few reports of data to download. The PDF Wizard allows you to consolidate a range of data including glucose, insulin and CGM reports.
- 3 Select the desired time interval for each data set you have chosen. You may set independent time periods for each data set chosen.
- 4 When you are finished selecting your data and time intervals, scroll to the bottom of the window and click the Create Report button.
- 5 You can also click on the PDF icon in the Patient List to create your compiled PDF report or click on the downward pointing arrow to select the pre-set profile you wish to view/print.
- 6 The PDF Wizard can also be found under Settings in your account. There you can also add new customised report profiles.

The screenshot shows the 'PDF Wizard' form. Callout 1 points to the 'End date of time interval' field, which is set to '2014-09-11'. Callout 2 points to the 'Color' radio button, which is selected. Callout 3 points to the 'Time interval' dropdown menus for Glucose, CGM, and Insulin, all of which are set to 'Two weeks'.

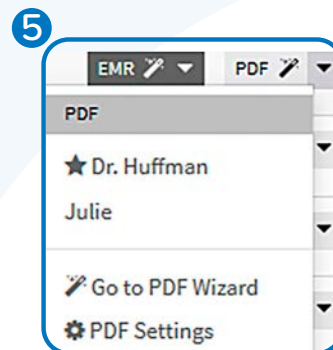

The screenshot shows the 'EMR/PDF Wizard' settings page. Callout 6 points to the 'Favorite Reports Profiles' table. The table has columns for 'Name of Profile', 'Default', and 'Action'. It lists profiles for 'Dr. Huffman' and 'Julie'. Below the table is a section for 'Action' with radio button options for different behaviors.

Name of Profile	Default	Action
	<input type="radio"/>	<input type="button" value="Edit"/> <input type="button" value="Delete"/>
Dr. Huffman	<input checked="" type="radio"/>	<input type="button" value="Edit"/> <input type="button" value="Delete"/>
Julie	<input type="radio"/>	<input type="button" value="Edit"/> <input type="button" value="Delete"/>

For more information on diasend by Glooko please contact us on 09 259 4062 or at [diabetes@nzms.co.nz](mailto:diabetes@nzms.co.nz)

[nzmsdiabetes.co.nz](http://nzmsdiabetes.co.nz)



To contact Glooko directly, visit [glooko.com/int](http://glooko.com/int) or email [support@diasend.com](mailto:support@diasend.com). NZDI\_DIA\_004 May 2020