


Personal Profiles

for the t:slim X2 Insulin Pump




The instructions below are provided as a reference tool for caregivers who are already familiar with the use of an insulin pump and with insulin therapy in general. Not all screens are shown. For more detailed information on the operation of your Tandem insulin pump, please refer to its user guide.

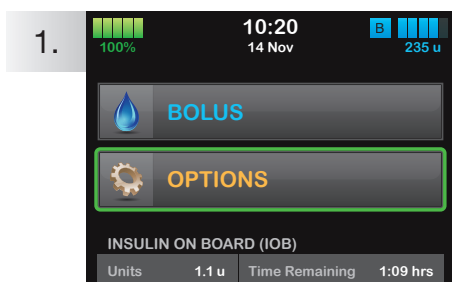


1.

Tap **OPTIONS**.

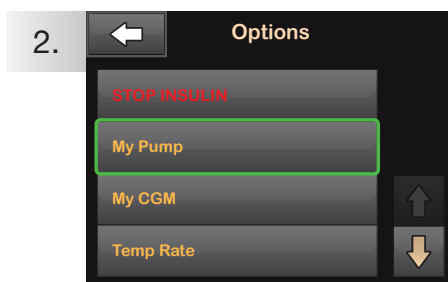


2.

Tap **My Pump**.

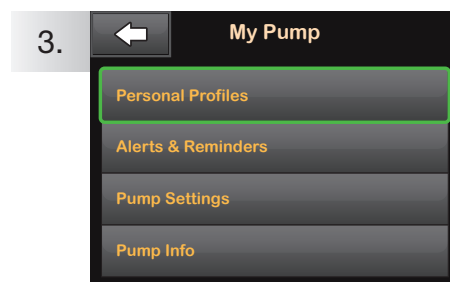


3.

Tap **Personal Profiles**.

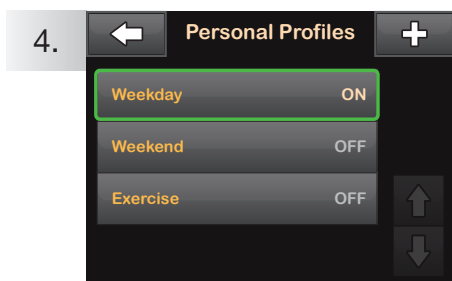


4.

Tap the name of the Personal Profile to view or edit.

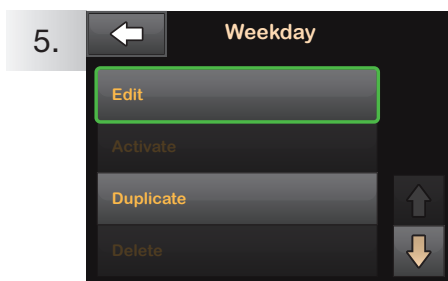


5.

Tap **Edit** to edit or view your settings.

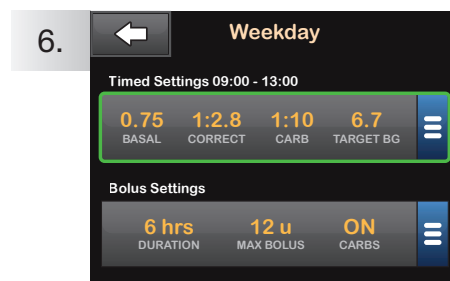

6.

Tap your current settings to see the other segments of your day.

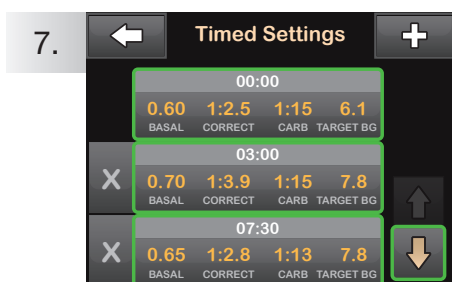

7.

Tap the time segment you wish to edit.

If not all segments are visible, tap the **Down Arrow**.



8.

Tap **Basal**, **Correction Factor**, **Carb Ratio**, or **Target BG** to make changes, then tap .

When you are finished, tap .



9.

Confirm settings. Recent changes appear in orange.

Tap to confirm.



t:simulator App
A demo as simple as our pumps!



© 2019 Tandem Diabetes Care, Inc. All rights reserved. Tandem insulin pumps are intended for the subcutaneous delivery of Humalog or NovoRapid/NovoLog U-100 insulin, at set and variable rates, for the management of diabetes mellitus in persons requiring insulin. The t:slim X2™ insulin pump is approved for individuals 6 years of age and older. For important safety information, please visit www.tandemdiabetes.com/safetyinfo. Tandem Diabetes Care is a trademark of Tandem Diabetes Care, Inc., registered in the U.S. and other countries. t:slim X2 is a trademark of Tandem Diabetes Care, Inc. Apple and the Apple logo are trademarks of Apple Inc., registered in the United States and other countries. App Store is a service mark of Apple Inc. Google Play and the Google Play logo are trademarks of Google LLC. Tandem is distributed in New Zealand by NZMS Diabetes, 2a Fisher Crescent, Mt Wellington, Auckland 1060. Ph: 09 259 4062 . April 2020 NZDI_TAN_007_B.

For more information on NZMS Diabetes product range, please contact us on **0800 500 226** or at nzms@nzms.co.nz

nzmsdiabetes.co.nz

